

Let's make sure you have everything you need before testing begins! Double check your supplies to make sure they haven't expired.

Collection supplies include:


You will have one of the following supplies for each student being tested. We'll also include gloves and alcohol swabs.

- **Barcode Label** (We've included an extra just in case.)
- **Nasal Swab**
- **Test Tube**
- **Absorbent Pad**
- **Specimen Bag**

You should have enough kits in each box to test 10 students. Extra supplies are provided to the school.

You'll need to provide:

- **Ink Pens**
- **Hand Sanitizer**
We recommend each student sanitize after testing.
- **Surface Cleaner**
We recommend cleaning the surface of your station before testing.

 NOTE: Do not let bleach come into contact with the viral transport media liquid because it could become a chemical hazard. Let any disinfectant cleaner on surfaces dry completely before testing begins.

Now, you're ready to start testing!

1 Pass out the kits and help your students fill out the barcode with their first name, last name and date of birth. Have them **place the barcode length-wise on the test tube**. If the barcode is wrapped around the tube, we won't be able to read it.

3 Guide students as they insert the entire tip (less than 1 inch) of the swab inside their nostril and **rotate it in four circles against the inside of the nose**. Repeat this process in the other nostril with the same swab.

TIP: This process should take 15 seconds. For young students, this is a great time to practice counting to 15 or singing the A-B-C's!

2 Next, students can unwrap their swabs from the packaging. You might need to help younger students the first time. Remind them **not to lay their swabs down or let them touch any surfaces**.

4 Students can **insert their swabs, tip-first, into their individual tubes**. Then, they need to **break off the excess handle at the line**, and close the cap. Younger students might need a little extra help the first time.

Time to pack up!

Scan each student's barcode label before they place their tube in their individual bag with the absorbent pad and seal it closed. Students can then place their bags back into the TestingWorks! box.

Now, it's time for the students to sanitize their hands and get back to class!

After you've tested your entire class, return the box to the centralized location for pickup and transport.

Test your entire class for COVID-19 in just a few minutes!

Let's make sure you have everything you need before testing begins! Double check your supplies to make sure they haven't expired.


Collection supplies include:

- **Gloves**
- **Alcohol Swabs**
- **Barcode Label** (We've included an extra just in case.)
- **Nasal Swab**
You should have enough for each student, individually packaged. Extras are provided to the school.
- **Test Tube**
You should have one tube per 10 students.
- **Absorbent Pad**
- **Specimen Bag**

You should have enough kits in each box to test 10 students.

You'll need to provide:

- **Ink Pens**
- **Hand Sanitizer**
We recommend each student sanitize after testing.
- **Surface Cleaner**
We recommend cleaning the surface of your station before testing.

 NOTE: Do not let bleach come into contact with the viral transport media liquid because it could become a chemical hazard. Let any disinfectant cleaner on surfaces dry completely before testing begins.

Now, you're ready to start testing!

1 Fill out the barcode label with a unique name per group of 10 students and **place the barcode lengthwise on the test tube**. If the barcode is wrapped around the tube, we won't be able to read it.

3 Guide students as they insert the entire tip (less than 1 inch) of the swab inside their nostril and **rotate it in four circles against the inside of the nose**. Repeat this process in the **other nostril with the same swab**.

TIP: This process should take 15 seconds. For young students, this is a great time to practice counting to 15 or singing the A-B-

2 Next, students can unwrap their swabs from the packaging. You might need to help younger students the first time. Remind them **not to lay their swabs down or let them touch any surfaces**.

4 With gloves on, open the cap of the tube and have your students **insert their swabs, tip-first**, one at a time. They'll need to **break off the excess handle at the line**, so all the swabs fit nicely with the cap on. Close the cap and wipe down with an alcohol wipe. Repeat process for each group of 10 students. **Now, it's time for the students to sanitize their hands and get back to class!**

Time to pack up!

Scan the barcode label before placing the tube into the specimen bag with the absorbent pad and seal it closed. Finally, pack everything up in the original TestingWorks! box.

After you've tested your entire class, return the box to the centralized location for pickup and transport to our lab.

Test your entire class for COVID-19 in just a few minutes!